Disposable Grasping Forceps

Instruction for Use

【Product name】 Disposable Grasping Forceps

[Product description]

Disposable grasping forceps is composed of jaw, outer tube, protecting cover and handle. The jaw is made of 06Cr19Ni10, the traveller and handle use ABS, the coat adopts PE.It can be classified into four kinds by the shape of the jaw. TY:oval-shape, ZJ: bevel-shape, 3ZX: three teeth-shape, 4ZX: four teeth-shape, 5ZX: five teeth-shape, CXDZ: alligator teeth-shape.

The Disposable Disposable Grasping Forceps sterilized using Ethylene Oxide. Shelf life is 2 years. Only single use.

[Specification and type]

ATE-YWQ-TY(ZJ, 3ZX, 4ZX, 5ZX)-18(23)X800(1200, 1600, 1800, 2300, 2500); ATE-YWQ-CXDZ-18(23)X800(1200, 1600, 1800, 2300, 2500)

【Usage】 The device is intended for use in combination with endoscope to remove foreign matters in the digestive tract.

[Method of use]

- 1. Take out foreign body forceps, from packaging for bent, broken, and so on and so forth.
- 2. The foreign body forceps completely withdraw the outer tube, and then insert the product into endoscopic biopsy channel, slow progress, until the foreign body forceps outer tube endoscope.
- 3. By endoscope or observed under fluoroscopy to remove target, will be pushed to the foreign body forceps outside tube more than the location of the target.
- 4. Confirm the foreign body forceps outside tube is placed in the right position, after the foreign body forceps will launch tube, slowly pull handle, show the goal.
- 5. Retreat foreign body forceps, until the foreign body resist endoscopic head, then put the endoscope from patient's body.
- 6. According to the regulations of the organization treatment of foreign body.

[Matters needing attention]

- 1. The operator of the use of this product must be a medical personnel, and must have the corresponding qualification;
- 2. This product by the EO sterilization, sterile products, valid for two years after the sterilization of qualified, recognizing when using single packing pincer-like device failure, overdue it is prohibited to use;
- 3. The single packing damage, spare parts, packaging banning the use of foreign body in it, and destroy;
- 4. This product is only for one-time use, destroyed after use.

Contraindications

- 1. Acute perforation of esophagus, stomach and duodenum.
- 2. Serious heart, lung, kidney, brain dysfunction and multiple organ failure. 3, psychosis, and impaired consciousness obviously can't collaborator.
- 3. And other specific by professional doctor according to the actual condition of the patient.

【Packaging and storage】

- 1. Single packaging uses paper-plastic package; middle packaging uses cartons; external packaging uses corrugated carton.
- 2. Products after packing should be stored in a well-ventilated room with relative humidity not more than 80% and no corrosive gas.

[Marking instruction]

Marking modecodorn			
Symbol	Title of Symbol	Symbol	Title of Symbol
			Authorized representative
	Manufacturer	EC REP	in the European
			Community
C € ₀₁₉₇	CE marking	{	Date of manufacture
REF	Catalogue number	LOT	Batch code
	Use-by date	STERILEEO	Sterilized using ethylene oxide
②	Do not re-use	\triangle	Caution



Working diameter



Logo



Jiangsu Ate Medical Technology Co.,Ltd.

No.8, Lanxiang Road, Wujin Economic Development Zone, 213161 Jiangsu, China

Tel: 0086-519-85288988 Fax: 0086-519-85288060



ZOUSTECH S.L.

Pso. Castellana, 141 - Planta 19, 28046 - Madrid, Spain

Tel: + 34694426446 Fax: + 34917915466

TÜV Rheinland LGA Products GmbH

Tillystraße 2, 90431, Nürnberg, Germany

Document number: ATE/IFU-YWQ-0001, version: A/1